
Intended Purpose: To transfer energy for defibrillation, cardioversion, therapy to the heart and provide CPR assistance/ feedback and ECG monitoring.

Indications for use: The CPR Dura-padz Reusable Defibrillation Electrode is used in conjunction with Dura-padz GEL on adult patients only with ZOLL AED Pro®, X Series® and R Series® by trained personnel only including Physicians, Nurses, Paramedics, Emergency Medical Technicians, Cardiovascular Laboratory Technicians, and First Responders.

The CPR Dura-padz Reusable Defibrillation Electrodes are not indicated for use on a patient less than 8 years of age or weighing less than 55lbs (25kg).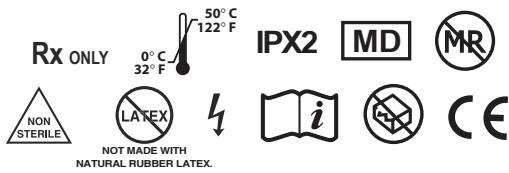

TEMPERATURES

OPERATING TEMPERATURE: 0° C to 50° C (32° F to 122° F)

SHORT TERM STORAGE TEMPERATURE: -30° C to 65° C (-22° F to 149° F)

LONG TERM STORAGE TEMPERATURE: 0° C to 35° C (32° F to 95° F)

SERVICE LIFE

CPR Dura-padz Reusable Defibrillation Electrode

Not recommended for more than 100 patient uses.

SERVICE LIFE Dura-padz GEL

Single patient use only.

PRECONNECTION

To prepare for emergency, keep CPR Dura-padz reusable defibrillation electrode plugged into defibrillator at all times.

⚠️ Do not open Dura-padz GEL pouch until ready for patient use.

SKIN PREPARATION

1. Remove excess chest hair to maximize gel-to-skin contact. Clipping is recommended.

2. Ensure skin is clean and dry under electrode. Remove any debris, ointments, skin preps, etc. with water (and mild soap if needed). Wipe off moisture/diaphoresis with dry cloth.

⚠️ Excessive hair can inhibit good coupling (contact), which can lead to the possibility of arcing and skin burns.

ZOLL Medical Corporation
269 Mill Road
Chelmsford, MA 01824-4105 USA
800-348-9011 Made in USA

ZOLL International Holding B.V.
Einsteinweg 8A
6662 PW Elst, Netherlands

ZOLL Medical Switzerland AG
Baarerstrasse 8
6300 Zug, Switzerland

ZOLL


ZOLL Medical Switzerland AG
Baarerstrasse 8
6300 Zug, Switzerland

ELECTRODE APPLICATION


1

Remove both Dura-padz electrodes from protective polyurethane sleeve.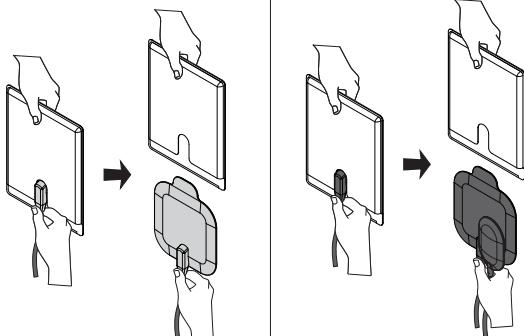


2

Locate Dura-padz GEL pouch and peel open as shown.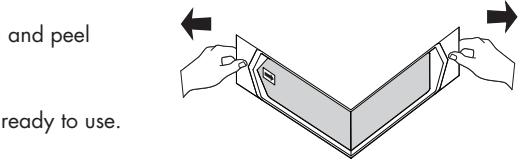

3



Open the pads like a book and place on flat surface. Remove yellow release liner only.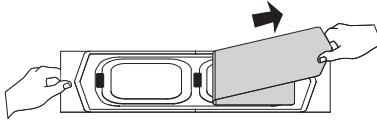

4


Visually align Dura-padz electrode over center of gel pad and place on top. Press firmly to secure entire area. Repeat with second electrode.

⚠️ Do not trap any air pockets between the electrode and gel.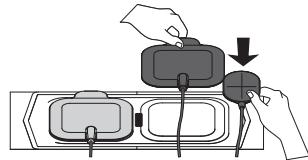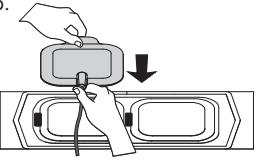


5

Grab the red tab and peel both the electrode and gel pad from the back release liner.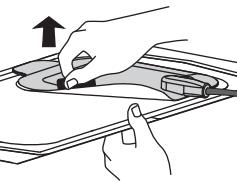

Apply to patient, then repeat with second electrode and gel pad set.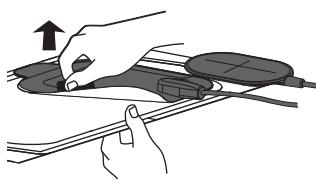



6

Apply one edge of the pad securely to the patient. "Roll" the pad smoothly from that edge to the other. Do not trap any air pockets between the gel and the skin.

⚠️ Poor adherence and/or air under electrodes can lead to the possibility of arcing and skin burns. Sternum: Apply electrode on upper right torso. CPR Sensor: Align CPR sensor with sternal notch.

Apex: Apply so that top of electrode aligns with bottom of pectoral muscle on male patient. Place under breast on female patient. The lateral placement increases the likelihood that more heart musculature will be within the current pathway.

⚠️ Avoid contact between nipple and gel area as it is more susceptible to burning.

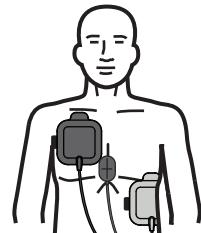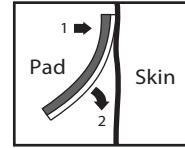CLEANING

After each use, remove **Dura-padz GEL** by handling red tab and peeling away from **CPR Dura-padz** electrode. Dispose of **Dura-padz GEL** per institution's protocol. Thoroughly clean **CPR Dura-padz** reusable defibrillation electrode using a gauze pad or soft cloth, and one of the following cleaning agents: 90% isopropyl alcohol, 3% chlorine bleach (30ml / L water), or 2% glutaraldehyde solution. Ensure all residue left from either conductive gel or adhesive has been removed, and allow cleaning solution to dry before returning to storage. Build-up of gel or other contaminants on the metal surfaces can lead to corrosion, discoloration, and other forms of degradation.

Do not immerse device in water. Do not use ketones (MEK, acetone, etc.). Do not sterilize device.

STORAGE

After cleaning, store **CPR Dura-padz** electrodes in protective polyurethane sleeve provided by sliding electrodes and sensor into each of the outer pockets. Place at least one **Dura-padz GEL** unit into middle pocket for storage. Keeping the two components together will ensure product is ready for immediate use. Rest on top of defibrillator with electrode cable loosely coiled. Do not wrap cable around the electrode or sleeve, as strain may be placed on device. Product is recommended for storage inside appropriate ZOLL carrying case.

INSPECTION / MAINTENANCE

Device must be regularly inspected, and maintained as needed to be ready for immediate use. Inspect the following items at the beginning of each shift:

CPR Dura-padz electrode: Verify that electrodes, sensor, cables, and connector are free of cracks, frays, cuts, and exposed or broken wires. Ensure all surfaces are free of any contaminants (i.e. dirt, gel, adhesive), and clean if needed.

Dura-padz GEL: Ensure pouches are properly sealed and undamaged. Verify product has not exceeded indicated expiration date.

Remove item(s) from service if any of the above criteria are not met.

USAGE LOG

Maintain record of each use by utilizing usage tally card attached to **CPR Dura-padz** electrode cable. After each patient use, punch out the next available marker. Repeat until all available markers have been removed. Remove **CPR Dura-padz** electrode from service after maximum of 100 uses.

⚠️ WARNINGS

1. Device is intended for use by trained personnel only including Physicians, Nurses, Paramedics, Emergency Medical Technicians, Cardiovascular Laboratory Technicians, and First Responders.
2. Not for use with ZOLL AED® Plus and/or any Public Access Defibrillators.
3. Use with ZOLL Biphasic Defibrillators only, Max Energy Setting: 200 Joules.
4. Do not store **CPR Dura-padz** electrode with metal surfaces in contact with one another, or with other metal objects.
5. Do not fold **CPR Dura-padz** electrode or **Dura-padz GEL** during storage or use, as it can result in damage leading to product failure, arcing, and/or skin burns.
6. Do not touch metal surfaces of **CPR Dura-padz** electrode. Contact during use may result in electrical hazard. Contact during storage may result in corrosion or other contamination that can compromise performance or lead to degradation of device.
7. Do not apply **CPR Dura-padz** electrode directly to patient, or attempt to use with defibrillator gels designed for use with paddles. Device must always be used in conjunction with **Dura-padz GEL**.
8. Do not use if **Dura-padz GEL** is dry. Dried out gel can lead to ineffective therapy, arcing, and skin burns.
9. To maximize patient benefit and minimize risk of burning, ensure **CPR Dura-padz** electrodes are properly aligned and secured to **Dura-padz GEL** before applying to patient.
10. Always apply electrodes to flat areas of skin. If possible, avoid applying over birthmarks, nipples, and skin folds such as those underneath the breast or those visible on obese individuals.
11. Avoid electrode placement near the generator of an internal pacemaker. Defibrillation may cause pacemaker malfunction.
12. To ensure proper adhesion, do not reposition electrodes once adhered to patient. If repositioning is required, use a new set of **Dura-padz GEL**.
13. To avoid electrical shock, do not touch electrodes, patient, bed, or other objects in contact with patient during defibrillation.
14. After patient movement due to muscle contraction or patient repositioning, press electrodes firmly to skin to ensure good coupling is maintained.
15. Do not perform chest compressions directly over electrodes, as this may cause damage to device resulting in ineffective therapy, arcing, and skin burns. Place hands directly on CPR sensor when conducting chest compressions.
16. Do not discharge standard paddles on or through device. Doing so could lead to ineffective therapy, arcing, and skin burning.
17. **Dura-padz GEL** should be replaced after 8 hours of wear to maximize patient benefit.
18. Removal of **Dura-padz GEL** from patient may result in temporary reddening of skin surface at adhesion site.
19. Defibrillation can result in skin burns.
20. Device disposal should follow hospital protocol.
21. Replace electrode assembly with appropriate pads if pacing is indicated.
22. Do not use electrodes in the presence of oxygen-rich environment or other flammable agents. Doing so could cause explosion.
23. If any serious incident has occurred in relation to the device, the incident should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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