

PERFORMING A RESCUE

1



Is patient responsive?

2




Call emergency services!

3



Get AED.

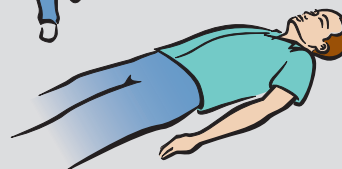

4




Place AED on ground.
Open lid.

5



Remove clothing.
Expose chest.

6

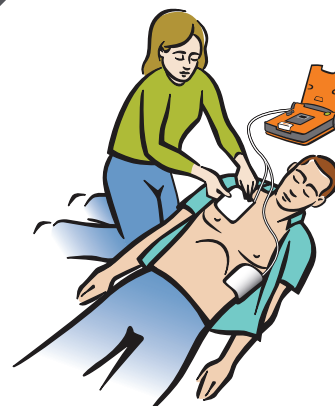

Attach pads.

7




Do not touch the patient.

8




If required, AED will deliver
shock automatically.

9

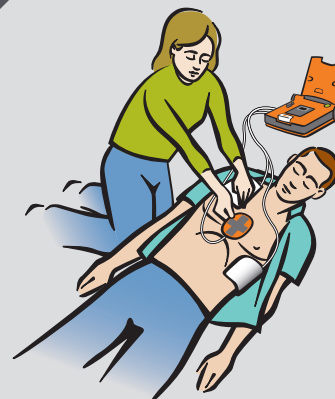

If present, place CPR device.

10

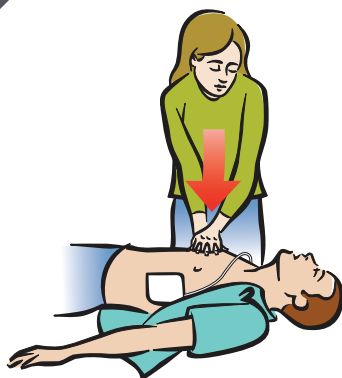

Follow AED prompts.
Give compressions.

11



If instructed, give breaths.

Automatic AED



Cardiac Science and the Shielded Heart logo are trademarks of Cardiac Science Corporation.
Copyright © 2019 All Rights Reserved.
500 Burdick Parkway, Deerfield, WI 53531 USA
www.cardiacscience.com



MDSS GmbH
Schiffgraben 41
D-30175 Hannover
Germany



70-02001-81 A



DÉROULEMENT D'UNE INTERVENTION

1



Le patient réagit-il ?

2




Appelez les services d'urgence !

3



Munissez-vous d'un DAE.

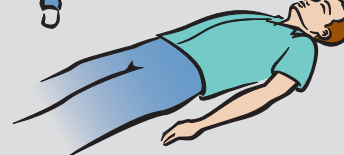

4




Posez le DAE sur le sol. Ouvrez le couvercle.

5



Enlevez les vêtements. Dénudez le thorax.

6

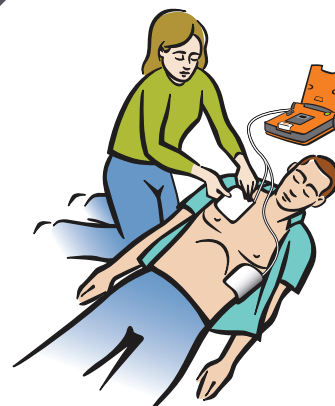

Placez les électrodes.

7




Ne touchez pas le patient.

8




Si nécessaire, le DAE administre automatiquement le choc.

9

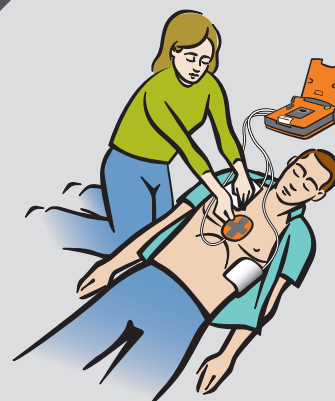

Si un appareil de RCP est disponible, positionnez-le.

10

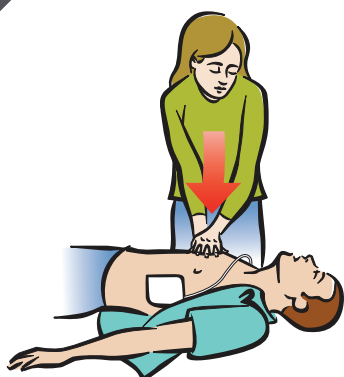

Suivez les invites du DAE. Administrez les compressions.

11



Si le système vous y invite, administrez les insufflations.

DAE automatisé



Cardiac Science et le logo Shielded Heart sont des marques commerciales de Cardiac Science Corporation.
Copyright © 2019. Tous droits réservés.
500 Burdick Parkway, Deerfield, WI 53531, États-Unis
www.cardiacscience.com



MDSS GmbH
Schiffgraben 41
D-30175 Hanovre
Allemagne



70-02001-81 A

