

10 If no shock advised or after shock delivered, "Start CPR"

1 Turn AED on

During chest compressions, the ZOLL AED 3 will provide Real CPR Help® prompting for CPR compression quality

2 "Check responsiveness"

3 "Call for help"

4 "Open airway"

5 "Check breathing"

6 "If patient is a child, press Child button"

7 "Attach pads to patient's bare chest"

8 "Don't touch patient. Analyzing"

9 If shock advised, "Press flashing Shock button"



For more information on the ZOLL AED 3, visit us at zoll.com/aed3.

ZOLL®