

RescueNet Code Review Install Guide and Hardware/Software Specification

SOFTWARE VERSION 5.9.0 AND NEWER

April 2, 2019



© 2019 ZOLL. All rights reserved.

ZOLL is a registered trademark of ZOLL Medical Corporation.

Other product and company names may be the trademarks of their respective owners.

Table of Contents

Overview	1
Intended Audience	1
How to use this document	1
Installation Prerequisites	1
RescueNet Code Review Standard vs. Enterprise Edition	2
Installation Folder Requirements	2
Installing RescueNet Code Review	2
Getting Help	10

Overview

This document describes the steps required for a successful RescueNet Code Review installation. The installer must complete all configuration items described in this document to ensure a functioning system.

Intended Audience

This document is intended for all installers. To familiarize yourself with the steps and items you need to complete the install, read the entire guide before beginning the install.

How to use this document

This guide is divided into sections in the order that the tasks should be performed. The installer must perform the tasks to ensure a successful installation.

Installation Prerequisites



Important! RescueNet Code Review versions preceding 5.8.1 must be manually uninstalled prior to the installation of 5.9.0.

RescueNet Code Review supports Microsoft Windows 7, Windows 8.1, Windows 10, as well as Windows Server 2008+. Both 32-bit and 64-bit operating systems are also supported. Any operating system other than those listed are not supported. Installing RescueNet Code Review on an operating system other than those listed may lead to an inoperable system and will not be supported by ZOLL Data Systems.

RescueNet Code Review requires .NET 3.5 and the full Microsoft .NET Framework 4.0. Ensure that they are installed prior to running the install. This redistributable package is available for download from ZOLL's website <http://www.zolldata.com/c-r-downloads/> or www.microsoft.com.

RescueNet Code Review requires Windows Management Framework 3.0+ and Microsoft Windows Installer 4.5 prior to starting the install. This install package is available at www.microsoft.com.

During the install, RescueNet Code Review installs Microsoft MSXML 4.0 sp3 and Microsoft Visual C++ 2013 x86 automatically.

RescueNet Code Review Standard vs. Enterprise Edition

RescueNet Code Review Standard Edition can be installed without a license key.

RescueNet Code Review Enterprise Edition requires a valid license key. This edition provides access to view 12-Leads and Vital Sign data that could be recorded by the defibrillator as well as additional data entry fields. For more information, see the RescueNet Code Review Getting Started Guide.


Installation Folder Requirements

The default RescueNet Code Review installation folder is C:\Program Files (x86)\ZOLL Data Systems.


RescueNet Code Review requires a Windows Media Player. This feature must be enabled in the Control Panel.

Installing RescueNet Code Review


1. Launch the setup wizard by clicking “Code Review – 5.9.0.xxx”.
2. Click Run

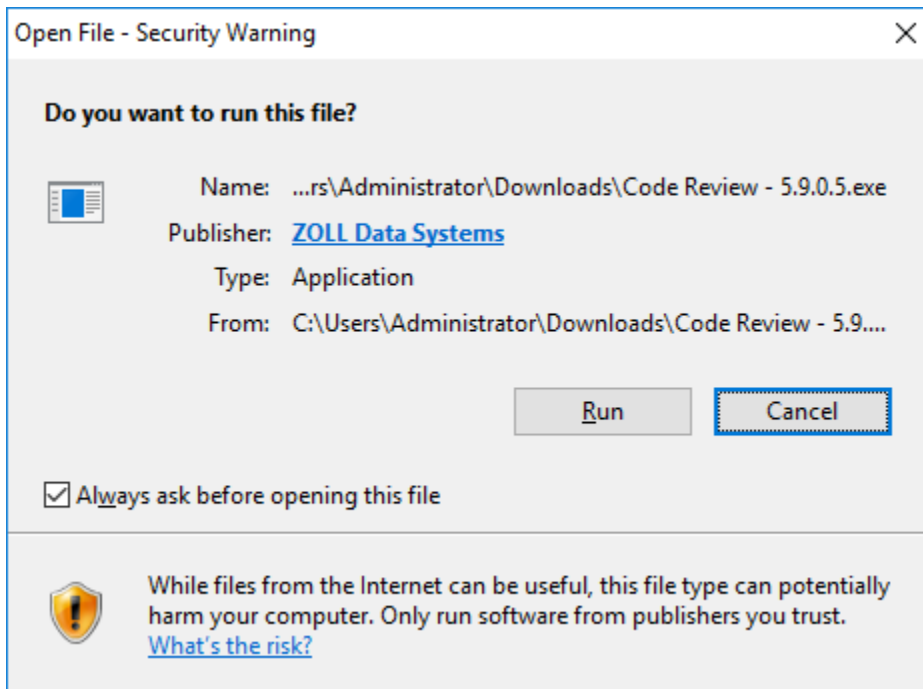


3. Click Install.

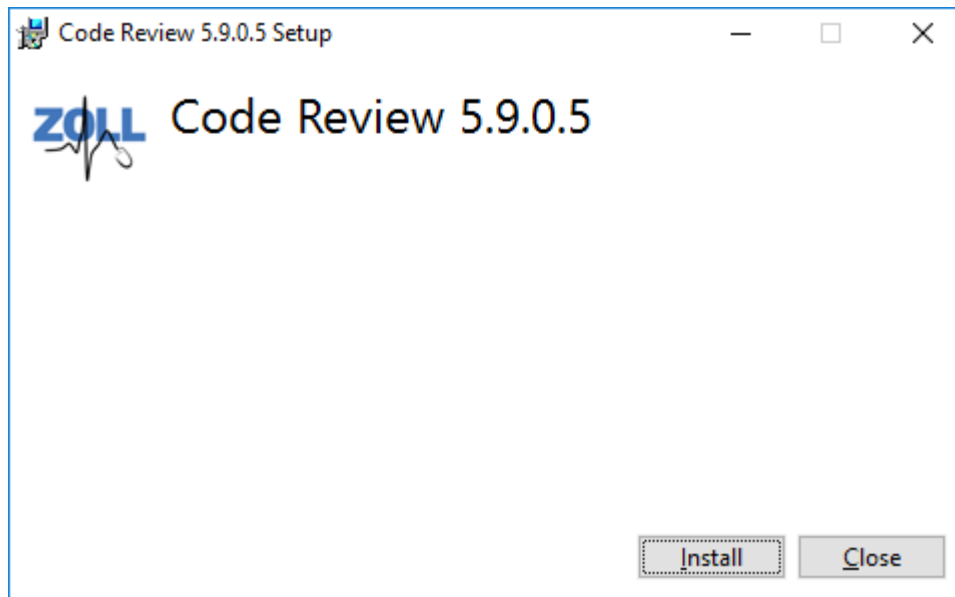


The installer will install some prerequisites, including version 6.43.8 of the SDK.

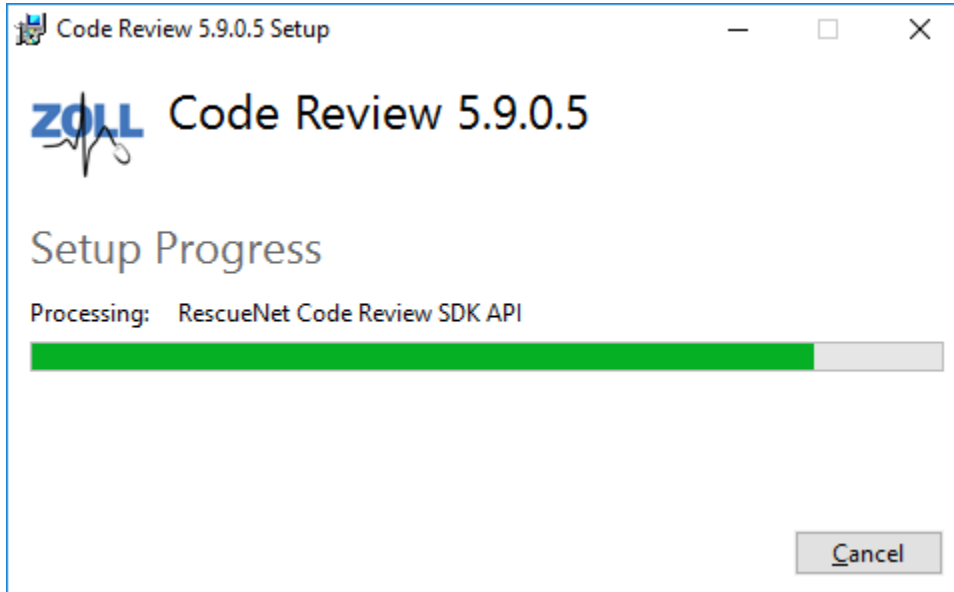

4. Once prerequisites have been installed, the Code Review installer welcome page will display. Click Next.

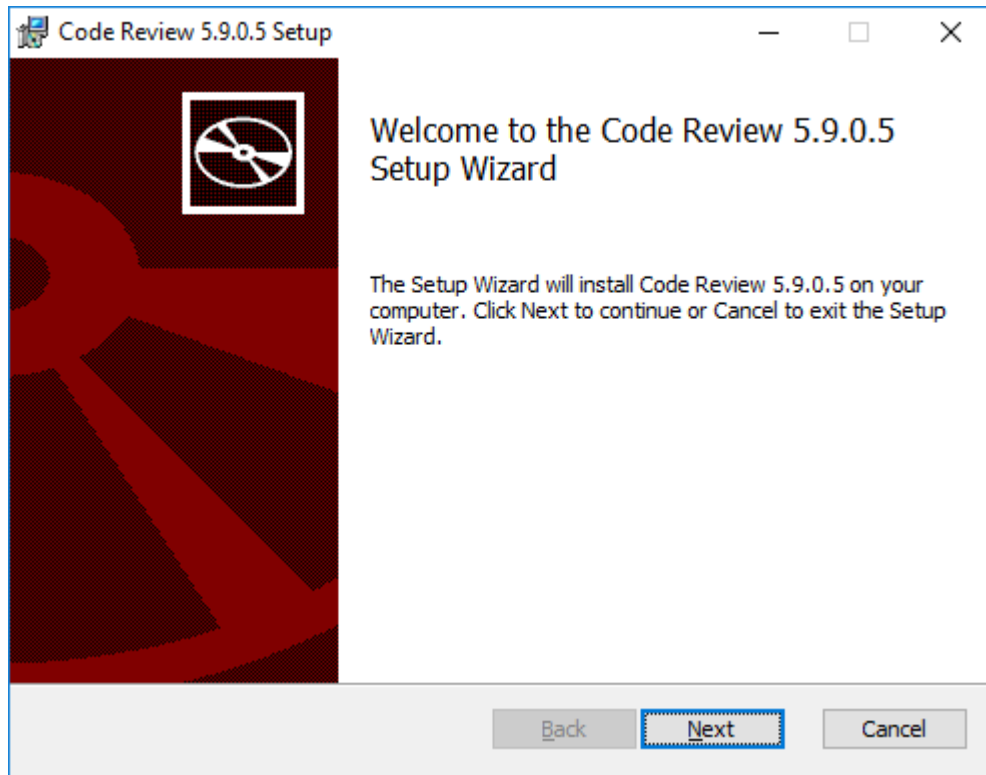


5. Accept the License Agreement and click Next.

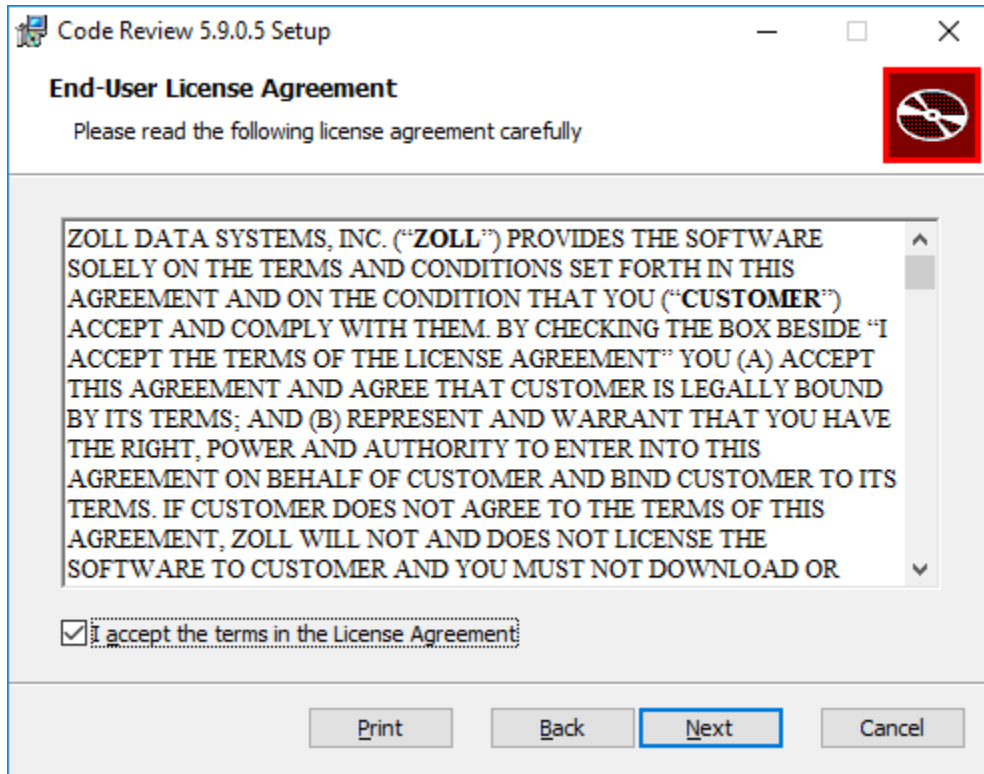


6. Select the folder to install tol and/or select a language version to install from the Language Selection drop-down and click Next. We recommend that you accept the


default installation folder in most cases.

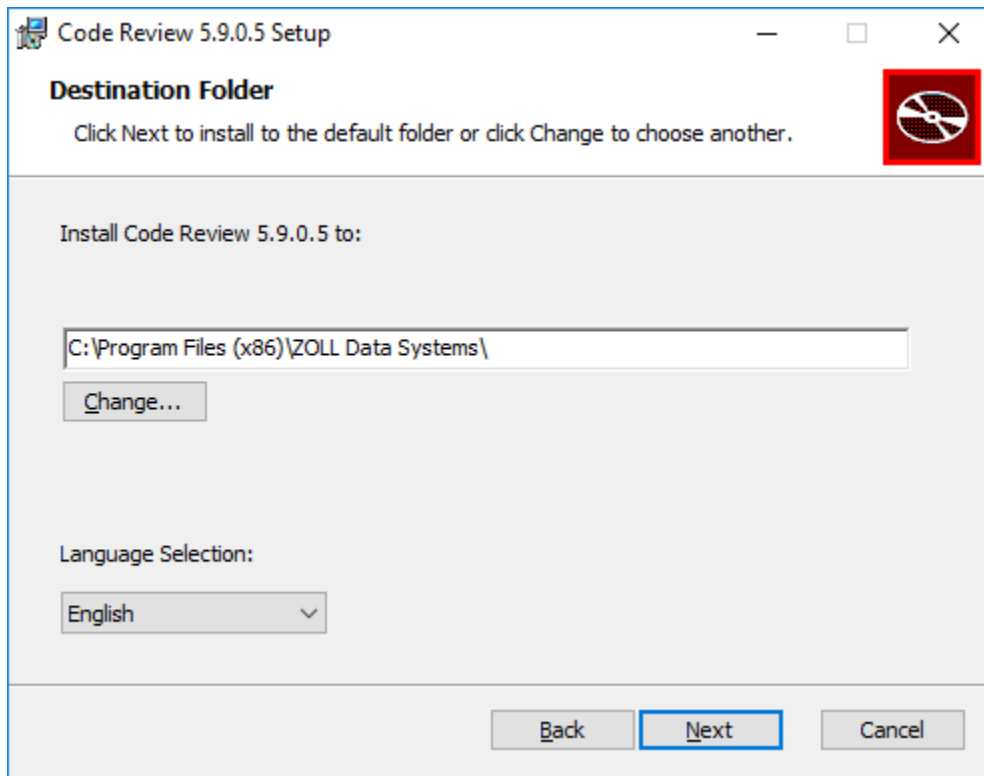


7. You are now ready to install Code Review. Click Install to continue.

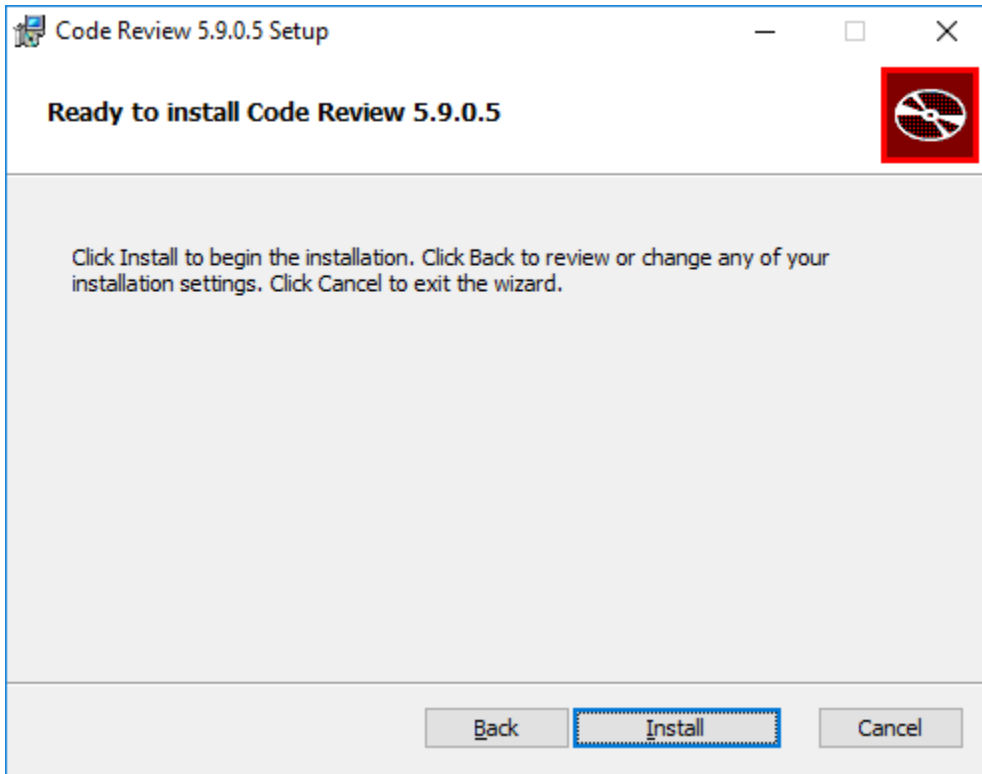

- Once finished installing, you will be shown the Completed dialog. Click Finish to complete the setup.

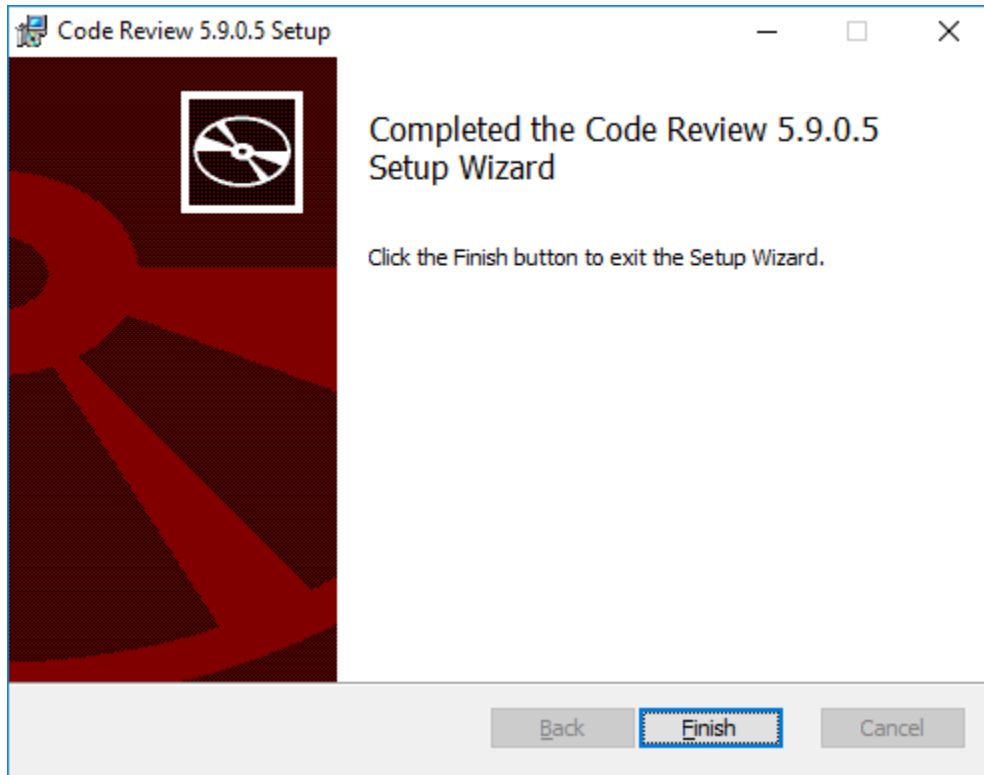

- Installation is successfully completed.

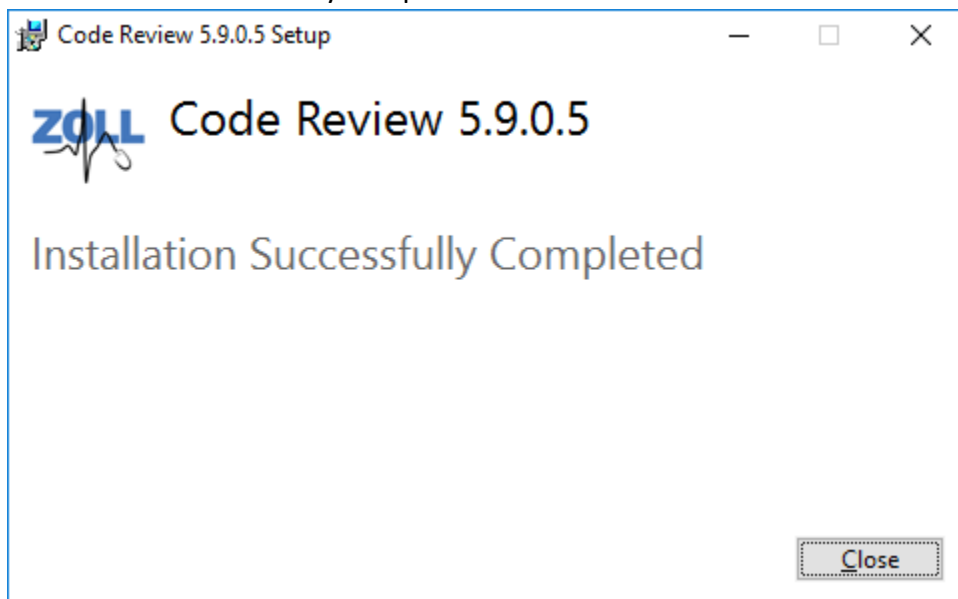

Getting Help

For customers in the United States, contact ZOLL Medical Corporation.

ZOLL Medical Corporation
269 Mill Road
Chelmsford, MA 01824-4105 USA
(800)348-9011 or (978)421-9655
Supportdata@zoll.com
Fax: (978)421-0015
www.zoll.com

For customers in all other locations, send email to Supportdata@zoll.com or contact your authorized local ZOLL representative.

International offices use <http://www.zoll.com/contact.htm>.