

# RescueNet Code Review™ Release Notes

SOFTWARE VERSION 5.9.0

April 1, 2019



© 2019 ZOLL. All rights reserved.

ZOLL is a registered trademark of ZOLL Medical Corporation.

Other product and company names may be the trademarks of their respective owners.

## Code Review 5.9.0

The 5.9.0 release of Code Review is concerned primarily with adding the ability to display data from a ZOLL M2 device.

### Work Items

ID	Release Notes
153743	Fixed an issue where Code Review could not be installed if a newer version of the SDK was already installed on the computer. Now Code Review will continue to install, and the most recent version of the SDK is preserved on the system.
183497	Fixed an issue where silent install (using "/install /quiet /norestart" flags) would result in an error.

## Code Review 5.8.1

### Work Items

ID	Release Notes
4193	QRS markers shown on ECG strips were not printing on the Full Disclosure Report.
29805	The "Life threat alarm" event was incorrectly listed twice in the event tree. When clicked, it was not displaying the snapshot on the Snapshot tab, but instead displayed an error.
29890	The "No breath alarm" event times were off by one second compared to the X Series Treatment Summary Report.
30229	Because Code Review cannot print large case files, it now alerts users to loss of functionality when a case file larger than 75mg is opened.
32318	The "Life threat alarm" event has been removed from Event Summary options when printing Strips for a case.
32326	Code Review now displays two events for each "No breath alarm" that occurs in a case: a "No breath alarm" and a "No breath alarm - breathing resumed" or "No breath alarm - breathing not resumed," so that the alarm and the resolution are shown in two separate events.

39930	Code Review has been translated into Simplified Chinese for tenants installing in that language.
39931	Code Review has been translated into Brazilian Portuguese for tenants installing in that language. <span style="float: right;">Page 2</span>
45181	When adding a CPR Pause Period, Code Review was not allowing the user to save a reason for a pause period of type "Other."
45182	The functionality to adjust a CPR pause period's start and end times using numeric time entry has been added back to the CPR pause period dialog box.
45229	Opening a case from an AED Plus device no longer gives the user an error and displays the file with corrupted fields.

48574	In non-English versions of Code Review, the scroll bar on the Magnified ECG tab was incorrectly located above the ECG grid, instead of below it.
54511	The user may now install Code Review in any location, not just the root directory.
60189	Code Review logic has been updated to reflect the CPR periods created by the newest release of the AED 3.
63784	The AED3 device prompt "Attach pads to child's back and chest" now displays in Code Review as a prompt in the Case Tree.
64502	For certain case files, the SpO2 waveform on the Magnified View tab was incorrectly showing a 100.0 - 0.0 scale for the y-axis.
77413	Users may upgrade prior Enterprise Editions of Code Review to release 5.8.1 without re-entering their original license key.